



Neo Dental International Inc.

Evadyne



LIGHT CURED TEMPORARY RESTORATION

Effectiveness

- **Evadyne** is a light cured temporary sealing material containing urethane resin and fluoride releasing ionomer glass.
- Quick and simple application directly into cavity.
- Convenient syringe allows direct placement without instrument for application.
- Tasteless, odorless and non-irritating material.
- Good sealing ability.
Easily removed in bulk with an explorer.

Technical Data

| | |
|---------------|------------|
| DEPTH OF CURE | 14.3 mm |
| FLEX STRENGTH | 3.4 MPa |
| CURING TIME | 20 seconds |




Manufactured by:
Neo Dental Chemical Product Co. Ltd.
Tokyo, Japan



Neo Dental International Inc.

2505 S. 320th ST Suite 625
Federal Way, WA 98003, USA

Tel: 253-946-0814
Fax: 253-946-0885
www: neodental-intl.com
email: ndi@neodental-intl.com

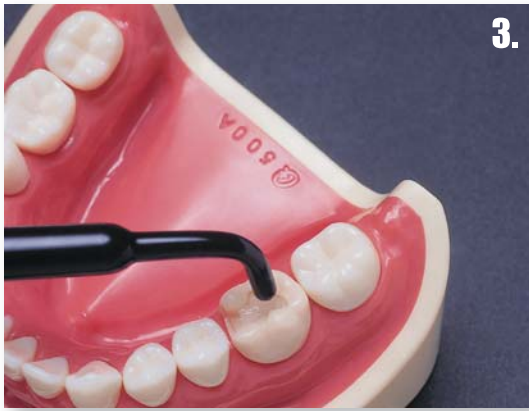

Directions

1. Rinse and dry prepared cavity.
2. In cases with direct pulp exposure apply protective base of calcium hydroxide.
3. Line cavity with liner.
4. Directly fill the cavity with **Evadyne**. Contour the occlusal surface to produce a slightly concave shape.
5. Wet opposing teeth before adjusting occlusal surface.
6. Light cure for 20 seconds (or 10 seconds in a shallow cavity) with a standard curing light.
7. Discard tip after each patient. Tightly place cap on syringe. Use a new tip for each patient.
8. **Evadyne** can be removed in one piece using a hand instrument.

Caution

- Do not use **Evadyne** on patients with a history of allergy or irritation to **Evadyne** or methacrylates. If irritation, stop using and consult a physician.
- Evadyne should not come in contact with the soft tissue in the mouth, skin or eyes. If eye contact does occur, immediately irrigate with water or saline solution and see an ophthalmologist.
- Use **Evadyne** only as prescribed.

Storage

- Avoid exposure to direct sunlight.
- Storing Evadyne under cool conditions extends the shelf life.

Packaging

- 5 g / syringe
- 15 disposable tips



Neo Dental International Inc.

2505 S. 320th ST Suite 625
Federal Way, WA 98003, USA

Tel: 253-946-0814

Fax: 253-946-0885

email: ndi@neodental-intl.com